USER'S MANUAL

1. Device:

Salt pipe family (ceramic and plastic)

2. Manufacturer:

Sópipa Pharma Kft. – 1146 Budapest, Francia u. 57/c. - Hungary

3. Area of use:

Indications

- Respiratory tract allergy,
- Asthma,
- Bronchitis,
- Chronic obstructive pulmonary disease,
- Cold,
- Breathing problems persisting after quitting smoking.

Contraindications

- No known contraindication.

Planned user:

Anyone can use it from the age of 3. Children can only use it under parental supervision.

4. Risks

Based on the evaluation of the risks associated with the use of the device, please take the following into consideration when using the device:

- Do not use a damaged or broken device.
- Do not use a contaminated or dirty device.
- The cleaning of the device should be performed using a mildly damp cloth on the outer surface of the device.
- Attention, no water or liquid can get into the device as it damages the salt filling.
- Only one person can use one salt pipe.
- In case of prolonged, deep, forced breathing, a salty taste in the mouth, dizziness or nausea may occur.

5. Cleaning the device

The cleaning of the device should be performed using a mildly damp cloth on the outer surface of the device. Make sure no liquid gets into the device, as it may lead to the coagulation of the salt filling.

6. Warnings

The cleaning of the device should be performed using a mildly damp cloth on the outer surface of the device. Attention, no water or liquid can get into the device, as it damages the salt filling.

Store in a dry place without humidity.

Only one person can use one salt pipe. (To avoid possible infections!)

In case of prolonged, deep, forced breathing, a salty taste in the mouth, dizziness or nausea may occur.

7. Designed duration:

For ceramic salt pipe: 5 years – For plastic salt pipe: 1 year

8. Storage and transportation conditions:

The transportation of the device is recommended in its original packaging. Store the product in a dry place without humidity.

9. Use of the device:

Take the device into your mouth and breath through it, then exhale the air through your nose and keep a normal breathing intensity throughout. Forced breathing is useless and should be avoided. When the cleaning process starts, the depositions may leave the body through coughing and a running nose.

The salt pipe is recommended to be used for 15–25 minutes a day. Treatments may be performed even multiple times a day.

In order to reach a beneficial result, the device should not be used seasonally, but continuously.

The salt crystals induce the mucous membranes of the respiratory tract to perform intensive cleaning, thereby helping to launch self-cleaning processes.

10. Composition of the salt filling:

Sodium-chloride: 99% (min. 97%)

Other components: 1% (max. 3%) – Calcium-sulphate, magnesium-chloride, calcium-chloride, iron (III)-oxide.

11. Meaning of the applied symbols

Manufacturer

Certification mark

Batch code, the device's

series ID Warning

Keep dry

Read the user's manual

12. Identification of the user's manual

Version of the user's manual: 3, date of release: 18.12.2019

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